**materialise**

innovators you can count on

# Eyewear Mobile 1.0

## Instructions for Use

Included languages:

- Deutsch
- English
- Français
- Nederlands

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Manufactured in 2022-02 by:  
Materialise NV  
Technologielaan 15  
3001 Leuven, Belgium

## Instructions for Use

### *Eyewear Mobile 1.0*

This document contains the basic Instructions for Use for Eyewear Mobile.

#### Description

Eyewear Mobile offers size selection of (3D printed) eyewear. Based on a set of standard sizes for the front of the frame and the temples, Eyewear Mobile will select the best available size of front and temples to match the customer's anatomy.

The application receives input from a 3D scanning device and reconstructs this input information into a 3D model. Eyewear Mobile will then fit all available sizes of the frame on the 3D model and select the best fitting size for the front of the frame (A-size and width of the nose bridge) and the temples based on anatomical fit and frame position. The frame can then be ordered through an online web service. The frame positioning information from the software can also be used to order lenses through an external system.

#### Intended purpose

Eyewear Mobile is intended to be used for the size selection of (3D printed) optical frames.

Eyewear Mobile will fit all available sizes of the frame on a 3D scan and select the best fitting size for the front of the frame (A-size and width of the nose bridge) and the temples based on anatomical fit and frame position. If more than one size is suitable for the customer, the user can still choose one of the additional options. Manufacturing of the frame can then be ordered. The frame positioning information from the software can also be used to order lenses through an external system.

Eyewear Mobile should only be used by certified ophthalmologists, optometrists or opticians.

#### Warnings

To ensure safe operation of Eyewear Mobile and to protect the data managed by Eyewear Mobile, always run the application on an up-to-date version of iPad OS. Avoid access to customer data by limiting access to the system by automatically locking the screen when the tablet is not in use.

Only trained eyewear professionals may use Eyewear Mobile.

#### System requirements

Minimal: 12.9" iPad Pro 4th generation (March 2020)

Recommended: 12.9" iPad Pro 5th generation (April 2021)

OS: Minimal iPad OS 15.0

Network connection: permanent connection to the internet required

#### Installation and getting started

Install the Eyewear Mobile application from the Apple AppStore. On first startup, log in with the provided username and key on the login screen. The software will guide you through all the steps required to select an optical frame and order it.

#### Legend of symbols used in the about box

Date of manufacture

Batch code

Legal manufacturer

Unique Device Identifier

Medical device

Caution, please consult the instructions for use and the accompanying documentation

Consult (electronic) instructions for use

#### Contact details

For any additional information, contact Materialise NV at:

Materialise NV  
Technologielaan 15  
3001 Leuven  
Belgium

[www.materialise.com](http://www.materialise.com)

+32 16 396 611

This is revision 01 of the document and has been issued in 2021-12.

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