CE  
0120

Manufactured by:  
Materialise NV  
Technologielaan 15  
3001 Leuven  
Belgium

## EN – Instructions for use

This document contains general instructions for use for the Zimmer® Patient Specific Instruments Planner 6.5.1 (Zimmer PSIP). For detailed instructions please refer to the user manual.

## DESCRIPTION

**Zimmer PSIP** is a 3D environment software which allows the surgeon to accurately plan a Total or Partial Knee Arthroplasty (TKA or PKA) based on MRI or CT scans of the patient. Once the plan is approved in the planner, Materialise will manufacture patient specific knee guides that will be used during the surgery.

## INDICATIONS

The **Zimmer PSIP** software is used for image processing, simulation of surgical interventions and the planning of pre-operational interventions.

The **Zimmer PSIP** is intended to be used to assist in the positioning of total and partial knee replacement components intra-operatively and in guiding the marking of bone before cutting and guiding cutting.

## CONTRAINDICATIONS

None

## WARNINGS

- This product should be used by trained technicians. The output must be verified by the responsible clinician.
- The 3D reconstructions must be compared to the slice images and verified for approval by the responsible clinician.
- It is the clinician's ultimate obligation to exercise his/her professional judgment in any decision to follow or not follow the treatment planning made using the medical device (software).
- Zimmer PSI are to be used by a trained physician in the performance of knee replacement surgery.
- Be aware that these patient specific devices and/or anatomical models derived from a planning created in the software have been manufactured based of MRI/CT scans of the patient. If the patient's anatomy has changed significantly since the time of the MRI/CT scan, the Zimmer PSI and/or models should not be used.

## PRECAUTIONS

- It is advised to use the Zimmer PSI or anatomical model within 6 months of performing the CT/MRI scans on which they are based. If the patient's anatomy has changed significantly since the time of the CT/MRI-scan, the Zimmer PSI or model should not be used, even if the time period of 6 months has not expired.

## MINIMAL REQUIREMENTS

### Software

- Windows 7
- Internet explorer 6.0

### Hardware

- CPU: Intel Pentium 4 – 2.4 GHZ or equivalent
- Memory: 512MB RAM
- Graphics card:
  - Minimal supported resolution: 1280x1024
  - Colors: 24-bit
  - Memory: 4.0 MB video memory
- Non-interlaced 15" color monitor
- Mouse that is supported by Windows
- A 56Kb internet connection

## RECOMMENDED REQUIREMENTS

### Software

- Supported OS: Windows 7, Windows 8.1, Windows 10
- Internet explorer 8.0 or higher

### Hardware

- CPU: Intel Core2 Duo 2.0 GHZ or equivalent
- Memory: 2GB RAM
- Graphics card:
  - ATI RADEON or NVIDIA GeForce
  - 128 RAM
  - Colors: 32-bit True Color
  - Memory: 128 MB video memory
- Non-interlaced 19" color monitor or 17" flat panel monitor
- Keyboard & optical mouse with wheel scroll
- High speed internet connection

## ADDITIONAL INFORMATION

For additional information, please contact your Zimmer representative and/or the Zimmer customer service.

### Note:

Zimmer and the Zimmer logo are trademarks of Zimmer, inc.

Materialise and the Materialise logo are trademarks of Materialise NV.

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*Zimmer* und das *Zimmer*-Firmenzeichen sind Warenzeichen von *Zimmer, Inc.*