materialise

innovators you can count on

Yuniku 7.0

Instructions for use

Included languages:

- Deutsch
- English
- Français
- Nederlands

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Manufactured in 12-2021 by:
Materialise NV
Technologielaan 15
3001 Leuven, Belgium

Instructions for Use

Yuniku software 7.0

This document contains the basic Instructions for Use of Yuniku software. Detailed instructions can be found in the Yuniku software manual.

Description

Yuniku software offers design of personalized optical frames (3D tailored eyewear). The optical frames are designed to position the lenses in a recommended position relative to the eyes (vision-centric design) and adapted to fit the anatomy of the patient.

The software receives input information from a 3D scanning device and reconstructs this input information into a 3D model and accepts user input (patient identification, lens prescription and information on the importance of daily activities the patient will use the glasses for) and recommended lens positioning information from the lens manufacturer. Based on this input, an automated frame design and lens positioning proposal is made, which can be fine-tuned by the user (frame position and dimensions can be modified). The frame and corrective lenses can then be ordered through an online web service.

Intended purpose

Yuniku software is intended to be used for the design of personalized optical frames including positioning of the corrective lenses.

Based on input from a 3D scanner, lens prescription, information on glasses use and recommended lens positioning information, an automated frame design and lens manufacturing information is generated. The user can review the proposed design and adjust frame position and dimensions when necessary. Manufacturing of the frame and the corrective lenses can then be ordered.

Yuniku software should only be used by certified ophthalmologists, optometrists or opticians.

Warnings

The Yuniku scanner used together with the Yuniku software needs to be calibrated before taking the first 3D scan. Re-calibrate the scanner regularly (every 2 weeks) or if you notice any of the following artefacts:

- Discontinuities of the 3D surface between the left and the right side of the face (middle of the nose, middle of the forehead, near the lips)
- Texture color differences or discontinuities between the left and the right side of the face (middle of the nose, middle of the forehead, near the lips)
- Holes or missing parts of the 3D surface

To ensure safe operation of Yuniku software and to protect the data managed by Yuniku software, always run the application on an up-to-date version of Microsoft Windows with all security updates installed and always enable antivirus software (like Windows Defender Antivirus) and a firewall (like Windows Defender Firewall). Avoid

access to customer data by limiting access to the system with a screen saver or lock screen.

Only trained professionals may use Yuniku software.

System requirements

CPU: Intel Core i7 processor recommended (Core i5 works as well, but computation-intensive tasks such as calibration, reconstructing the model and fitting will be slower)

Memory: minimally 8GB RAM, 16GB RAM recommended

Graphics card: Direct3D 11 compliant

Screen resolution: full HD (1920 × 1080)

OS: Windows 10 (a 64-bit OS version is required)

USB: 1 free USB3 port. USB3 ports are indicated with a blue connector inside or the SuperSpeed USB logo.

Network connection: permanent connection to the internet required

Installation and getting started

Yuniku software should be pre-installed on your computer during scanner installation. If this isn't the case, obtain the latest installer from the Yuniku product catalog at:

<https://portal.yuniku.com/#/downloadDesktopApp>

Start the installer package and follow the instructions on screen.

If you receive a security warning requesting approval to execute the installer, choose Run.

After completing the installation settings, accept the User Account Control notification to start installing Yuniku.

After the installation, Yuniku can be started automatically.

On first startup, you will be asked to log in to Yuniku. Use your existing practice account with Hoya. The software will guide you through all the steps required to fit a personalized optical frame. For more detailed instructions, consult the Yuniku software manual in the Settings – Manual section of the software.

Legend of symbols used in the about box

Date of manufacture

Batch code

Legal manufacturer

Unique Device Identifier

Medical device

Caution, please consult the instructions for use and the accompanying documentation

Consult (electronic) instructions for use

Contact details

For any additional information, contact Materialise NV at:

Materialise NV
Technologielaan 15
3001 Leuven
Belgium

www.materialise.com

+32 16 396 611

This is revision 07 of the document and has been issued in 12-2021.

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