**materialise**

innovators you can count on

# Materialise Eyewear 2.0

## Instructions for Use

Included languages:

- Deutsch
- English
- Français
- Nederlands

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Manufactured in 07-2021 by:  
Materialise NV  
Technologielaan 15  
3001 Leuven, Belgium

## Instructions for Use

### Materialise Eyewear 2.0

This document contains the basic Instructions for Use for Materialise Eyewear. Detailed instructions can be found in the Materialise Eyewear software manual included in the application.

#### Description

Materialise Eyewear offers size selection of (3D printed) eyewear. Based on a set of standard sizes for the front of the frame and the temples, Materialise Eyewear will select the best available size of front and temples to match the customer's anatomy.

The application receives input from a 3D scanning device and reconstructs this input information into a 3D model. Materialise Eyewear will then fit all available sizes of the frame on the 3D model and select the best fitting size for the front of the frame (A-size and width of the nose bridge) and the temples based on anatomical fit and frame position. The frame can then be ordered through an online web service. The frame positioning information from the software can also be used to order lenses through an external system.

#### Indications for use

Materialise Eyewear is intended to be used for the size selection of (3D printed) optical frames.

Materialise Eyewear will fit all available sizes of the frame on a 3D scan and select the best fitting size for the front of the frame (A-size and width of the nose bridge) and the temples based on anatomical fit and frame position. If more than one size is suitable for the customer, the user can still choose one of the additional options. Manufacturing of the frame can then be ordered. The frame positioning information from the software can also be used to order lenses through an external system.

Materialise Eyewear should only be used by certified ophthalmologists, optometrists or opticians.

#### Warnings

To ensure safe operation of Materialise Eyewear and to protect the data managed by Materialise Eyewear, always run the application on an up-to-date version of Microsoft Windows with all security updates installed and always enable antivirus software (like Windows Defender Antivirus) and a firewall (like Windows Defender Firewall).

Only trained eyewear professionals may use Materialise Eyewear.

#### System requirements

CPU: Intel Core i7 processor (Core i5 works as well, but computation-intensive tasks such as reconstructing the model and fitting will be slower)

Memory: minimally 8GB RAM, 16GB RAM recommended

Graphics card: Direct3D 11 compliant

Storage: SSD will allow to load models faster, but is not required

Screen resolution: full HD (1920 × 1080)

OS: Windows 10 (a 64-bit OS version is required)

USB: 1 free USB3 port. USB3 ports are indicated with a blue connector inside or the SuperSpeed USB logo.

Network connection: permanent connection to the internet required

#### Installation and getting started

Install the driver package for the Bellus 3D Face Camera Pro first:  
InuDriver\_3.0.7.0-1.07\_Setup\_X64.exe  
InuDev\_SP\_4.03.0012.85\_X64.exe

Next, run the Materialise Eyewear installer and follow the instructions on screen. After completing the installation settings in the installation wizard, accept the User Account Control notification to start installing the application.

After the installation the application can be started automatically.

On first startup, log in with the provided token through Menu > Account. The software will guide you through all the steps required to select an optical frame. For more detailed instructions, consult the software manual in the Menu > Manual section of the software.

#### Contact details

For any additional information, contact Materialise NV at:

Materialise NV  
Technologielaan 15  
3001 Leuven  
Belgium

[www.materialise.com](http://www.materialise.com)

+32 16 396 611

This is revision 01 of the document and has been issued in 07-2021.

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