# materialise

innovators you can count on

## SurgiCase Shoulder Planner 3.5

INSTRUCTIONS FOR USE - EN

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## INSTRUCTIONS FOR USE - EN

**This document contains general instructions for use for SurgiCase Shoulder Planner. For detailed instructions please refer to the user manual present in the application.**

### DESCRIPTION

SurgiCase Shoulder Planner allows the surgeon to simulate the placement of shoulder implants in a total (TSA) or reverse (RSA) shoulder replacement surgery. This simulation is based on three-dimensional models of the patient's anatomy that are derived from CT imaging data. Using the SurgiCase Shoulder Planner, the surgeon can inspect a default pre-operative plan that is generated by Materialise, adjust parameters as desired, and approve it. The software generates a surgery report along with a pre-surgical plan data file which can be used as input data to design a Materialise Shoulder Guide. This guide can be used on the patient during surgery as a guide to accurately determine drill and pin positions that are required for placing the actual implant device.

### INDICATIONS FOR USE

#### US

SurgiCase Shoulder Planner is intended to be used as a pre-surgical planner for simulation of surgical interventions for shoulder orthopedic surgery. The software is used to assist in the positioning of shoulder components. SurgiCase Shoulder Planner allows the surgeon to visualize, measure, reconstruct, annotate and edit pre-surgical plan data. The software leads to the generation of a surgery report along with a pre-surgical plan data file which can be used as input data to design the Materialise Shoulder Guide and Models.

#### EU and ROW

The SurgiCase Shoulder Planner software is intended to be used for simulation of surgical interventions and the planning of preoperational interventions.

The SurgiCase Shoulder Planner software can be used to assist in the positioning of orthopedic components.

### CONTRAINDICATIONS

None.

### WARNINGS

- The software should be operated by experienced personnel, or the output should always be reviewed and approved by a qualified surgeon.
- It is the surgeon's ultimate obligation to exercise his/her professional judgment in any decision to follow or not follow the treatment plan made using the medical device software.
- It is the surgeon's responsibility to only use the software on desktop or mobile devices satisfying the minimal requirements specified below.
- Be aware that the anatomical models shown in the software are derived from the images of the patient. If the patient's anatomy has changed significantly since the time of the images, the surgical plan should not be used.

### PRECAUTIONS

**R<sub>x</sub>only** CAUTION: Federal law (USA) restricts this device to sale by or on the order of a Physician.

### INSTRUCTIONS FOR USE

- Install  
No installation is required to use the SurgiCase Shoulder Planner. The SurgiCase Planner is a web based application and can be accessed through SurgiCase Online.
- Start

To open a case in the SurgiCase Shoulder Planner, browse to the appropriate case in the SurgiCase Online Ordering System and click the link in the Case Details to open the SurgiCase Shoulder Planner.

## MINIMAL REQUIREMENTS

### Operating Systems

Following operating systems are supported:

- Latest version of iOS
- Latest version of Mac OS
- Windows 10

### Browsers

Browser with WebGL support and WebGL enabled.

Following browsers are supported by the SurgiCase Shoulder Planner per operating system:

- Google Chrome on Windows, Mac OS X
- Microsoft Edge on Windows OS desktop
- Mozilla Firefox on Windows OS desktop
- Apple Safari on Mac OS X, iOS OSs

Cookies support

### Internet Connection

3G or higher

### Application window resolution

WXGA or higher

### Screen size

7 inches or more

### Memory

2GB RAM or more

### CPU

Dual Core processor or better

### Graphic card

Please refer to the following list describing black/white lists for graphic cards:

<https://www.khronos.org/webgl/wiki/BlacklistsAndWhitelists>

## Australian Sponsors

Ergo Australia Level 20 Tower II Darling Park

201 Sussex Street

Sydney NSW 2000

Australia

and

Mathys Orthopaedics Pty Ltd

PO Box 671, Lane Cove, NSW 1595

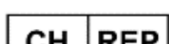
Australia

**CH authorised representative:**

MDSS CH GmbH  
Laurenzenvorstadt 61  
5000 Aarau  
Switzerland

**Description of symbols:**

	Legal Manufacturer
	Caution, please consult the instructions for use and the accompanying documentation
	Consult instructions for use or consult electronic instructions for use
	Date of manufacture
	Batch code
	Medical device
	Unique Device Identifier
	Authorised representative in Switzerland

IFU for SurgiCase Shoulder Planner is supplied in electronic form in PDF format on [www.materialise.com/electronic-instructions-for-use](http://www.materialise.com/electronic-instructions-for-use). Paper version may be requested by emailing [leaflet@materialise.be](mailto:leaflet@materialise.be) and will be provided within 7 calendar days at no additional cost.

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