materialise

innovators you can count on

Materialise Mimics Enlight 2.1

Instructions for Software Use – EN

Til brug af software – DA

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Instructions for Software Use - EN

This document contains the basic 'Instructions for Use' of Materialise Mimics Enlight Software. Detailed instructions can be found in the 'User Manual'. English is the language used in the Mimics Enlight software interface.

Description of Materialise Mimics Enlight software

Materialise Mimics Enlight is a workflow-based software interface for the transfer of medical images to an output file. The system provides visualization and measurement tools to enable the user to plan the procedure.

Materialise Mimics Enlight reads DICOM compliant medical images from a medical scanner, supporting a wide range of scanners. The software provides different methods to visualize and segment anatomical structures. Based on the medical images and/or 3D reconstruction measurements can be created and visualized. This all will be saved in a digital output file which can be afterwards used as input data for a CAD or additive manufacturing system.

US - Indications for use

Materialise Mimics Enlight is intended for use as a software interface and image segmentation system for the transfer of DICOM imaging information from a medical scanner to an output file.

It is also intended as a software to aid in reading and interpreting DICOM compliant images for structural heart and vascular treatment options. For this purpose, Materialise Mimics Enlight provides additional visualization and measurement tools to enable the user to screen and plan the procedure.

The Materialise Mimics Enlight output file can be used for the fabrication of physical replicas of the output file using traditional or additive manufacturing methods. The physical replica can be used for diagnostic purposes in the field of cardiovascular applications.

Materialise Mimics Enlight should be used in conjunction with other diagnostic tools and expert clinical judgement.

Materialise Mimics Enlight is not intended to predict the performance of a medical device.

Materialise Mimics Enlight is not intended for mammography imaging or diagnosis.

EU-ROW – Intended Purpose

Materialise Mimics Enlight is intended for use as a software interface and image segmentation system for the transfer of DICOM imaging information from a medical scanner to an output file.

It is also intended as a software to aid in reading and interpreting DICOM compliant images for structural heart and vascular treatment options. For this purpose, Materialise Mimics Enlight provides additional visualization and measurement tools to enable the user to screen and plan the procedure.

The Materialise Mimics Enlight output file can be used for the fabrication of physical replicas of the output file using traditional or additive manufacturing methods. The physical replica can be used for diagnostic purposes in the field of cardiovascular applications.

Materialise Mimics Enlight should be used in conjunction with other diagnostic tools and expert clinical judgement.

Materialise Mimics Enlight is not intended to predict the performance of a medical device.

Materialise Mimics Enlight is not intended for mammography imaging or diagnosis.

Best-practice tips and precautions

Mimics Enlight should only be used by trained medical professionals. To engage in the experience, we recommend that all users participate in a training offered by a Materialise expert.

The fluoroscopy tool can help plan C-arm angles; the planned angles will still need to be adapted during the procedure.

Further clinical investigations including a real clinical setting, linking the pre-operational planning with postop clinical data (neoLVOT area/blockage, survival) is needed to further confirm the clinical benefit.

Be advised that the quality of medical images determines the accuracy of the 3D model in Materialise Mimics Enlight. Scanning protocols are left to the discretion of the user; however, we recommend that industry standards are referenced and followed. Only images obtained less than six months before should be used for evaluating surgical treatment options.

Please be aware that the Dicom coordinate system is reset for each Dicom series originating from all Hitachi scanners and from Siemens Symphony, Sonata and Harmony scanners with software versions older than Syngo MR B13.

Materialise Mimics Enlight supports measurements with a precision of 0.01 mm/degree. Please note that the measurements are limited by the resolution of the images.

Rx^{only} CAUTION: Federal law (USA) restricts this device to sale by or on the order of a Physician

Instructions for use

Starting Materialise Mimics Enlight

After installing the software on your PC, double click the Materialise Mimics Enlight icon. You may also click on the **Start Button** → **All Programs** → **Materialise Software** → **Materialise Mimics Enlight** to begin.

Materialise software is protected by a key file. When you start Materialise Mimics Enlight for the first time or when your key has expired, the Key Request Wizard will automatically start up to assist you with the registration process. Available options are explained in the **User manual**. We recommend that you select '**instant activation**' in the Key Request Wizard. This will automatically renew your license Key File when it expires as long as you have internet access.

Minimum system requirements

Software	Hardware
Windows® 10 – 64bit Version 1909	Fourth generation Intel® Core™ i7 or equivalent
Microsoft Edge® or equivalent	16 GB RAM
PDF file viewer (.pdf)	DirectX® 11.0 compliant AMD Radeon/NVIDIA® GeForce® card with 2 GB RAM

.NET Framework 4.6.1 (or higher)	40 GB free harddisk space
	Resolution of 1920x1080

It is recommended to use our software within a hardware and/or network environment in which cyber security controls have been implemented including anti-virus and use of firewall.

Minimum system requirements virtual environment

The virtual set-up should meet the minimum system requirements. Additionally, AVX2 instructions set must be enabled.

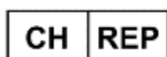
Description of symbols

	Legal Manufacturer
	Caution, please consult the instructions for use and the accompanying documentation
	Consult instructions for use or consult electronic instructions for use
	Date of manufacture
	Batch code
	Medical device
	Unique Device Identifier
	Authorised representative in Switzerland

Patent Notice

This product is covered by the following patents:

US9,943,373; US9,693,830

CH authorised representative:

MDSS CH GmbH

Laurenzenvorstadt 61

5000 Aarau

Switzerland

Materialise, the Materialise logo, Mimics and Enlight are trademarks of Materialise NV.

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