materialise

innovators you can count on

Mimics inPrint 3.0

Instructions for Software Use

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Description of Mimics inPrint software

Mimics inPrint is a software tool for visualizing and segmenting medical images (such as CT and MRI) and rendering 3D objects. Mimics inPrint may be used as a medical device, within the limits of the described below intended use statement.

Mimics inPrint may be used to load and process stacks of 2D images from numerous formats including: Dicom 3.0 format, BMP, TIFF, JPG and raw images. Once images are processed, they can be used for numerous applications of Engineering on Anatomy including measuring, designing, modeling and 3D printing.

Mimics inPrint enables users to control and correct the segmentation of medical images such as CT and MRI-scans. A trained software user can easily and accurately define 3D models for visualization and/or production.

Quantity of content = 1.

Indications for use

Mimics inPrint is intended for use as a software interface and image segmentation system for the transfer of DICOM imaging information from a medical scanner to an output file. It is also used as pre-operative software for treatment planning. For this purpose, the Mimics inPrint output file can be used for the fabrication of physical replicas of the output file using traditional or additive manufacturing methods.

The physical replica can be used for diagnostic purposes in the field of orthopedic, maxillofacial and cardiovascular applications. Mimics inPrint should be used in conjunction with other diagnostic tools and expert clinical judgement.

Contraindications

Please note that Mimics inPrint is *not* intended for mammography imaging.

Warnings and recommendations

Mimics inPrint software should only be used by trained professionals. We recommend that all users participate in a training offered by a Materialise expert for best results. Outputs intended to treat or diagnose patients must be checked by the responsible clinician who can compare the 3D reconstructions with the slice images to verify quality. It is the clinician's ultimate obligation to exercise his/her professional judgment as to whether the treatment planned using the medical device software (Mimics inPrint) should be performed. Inter-slice and in-slice distance of less than 1mm are recommended for treatment decisions based on 3D reconstructions for pre-operative planning or 3D printing. For bone related applications, we advise you to reconstruct the medical images in a window optimized for bone.

Mimics inPrint supports measurements with a precision of 0.01 mm. Please note that the measurements are limited by the resolution of the images.

Storage condition: no specific storage condition applicable for software.

Best-practice tips and precautions

Be advised that the quality of medical images determines the accuracy of the 3D model in Mimics inPrint. Scanning protocols are left to the discretion of the user; however, we recommend that industry standards are referenced and followed. Only images obtained less than six months before should be used for simulating and/or evaluating surgical treatment options.

Please be aware that the Dicom coordinate system is reset for each Dicom series originating from all Hitachi scanners and from Siemens Symphony, Sonata and Harmony scanners with software versions older than Syngo MR B13.

R_xonly CAUTION: Federal law (USA) restricts this device to sale by or on the order of a Physician

Instructions for use

Starting Mimics inPrint

After installing the software on your PC, double click the Mimics inPrint icon. You may also click on the **Start Button** → **All Programs** → **Materialise Software** → **Mimics inPrint** to begin.

Materialise software is protected by a key file. When you start Mimics inPrint for the first time or when your key has expired, the Key Request Wizard will automatically start up to assist you with the registration process. Available options are explained in the **Reference Guide**. We recommend that you select '**instant activation**' in the Key Request Wizard. This will automatically renew your license Key File when it expires as long as you have internet access*.

Minimum system requirements

Software	Hardware
Windows 7 – 64bit	Intel Core 2 Duo / AMD X2 AM2 or equivalent
Internet Explorer 8 and compatible	4 GB RAM
PDF viewer	DirectX 9.0c compliant graphics card with 512 MB RAM
	5 GB free hard disk space
	Resolution of 1280x1024

Note: Mac users can install Mimics inPrint using Bootcamp in combination with a supported Windows OS.

Note: Projects created or modified in a version prior to Mimics Medical/Research 18.0 are not supported by Mimics inPrint 2.0 and subsequent versions.

Preferred system requirements

Software	Hardware
Windows 7 - 64bit	Third generation Core i5/i7 or equivalent
Internet Explorer 8 and compatible	8 GB RAM
	DirectX 10.0 compliant ATI Radeon/nVidia GeForce card with 1GB RAM

	10 GB free hard disk space
	Resolution of 1680x1050 or higher

Note: Other qualifications may apply. When working with datasets larger than 1GB the system should comply with the recommended system requirements.

It is recommended to use our software within a hardware and/or network environment in which cyber security controls have been implemented including anti-virus and use of firewall.

Manufacturer contact details

Manufactured in March 2018 by:

Materialise NV

Technologielaan 15

B-3001 Leuven

Belgium

Phone: +32 16 39 66 11

<http://biomedical.materialise.com>

Australian Sponsor:

Emergo Australia; Level 20, Tower II Darling Park; 201 Sussex Street; Sydney NSW 2000; Australia

Thai Sponsor:

Mursmedic Thailand, 1687/1 Phahon 19 Building, Phahonyothin Road, Chatuchak, Chatuchak, Bangkok 10900, Thailand

CH authorised representative:

MDSS CH GmbH, Laurenzenvorstadt 61, 5000 Aarau, Switzerland

India Importer

Morulaa HealthTech Private Limited

No. 38, First Floor, Rajeswari Street, Santhosh Nagar,

Kandanchavadi, Chennai 600096, Tamil Nadu, India

Registration number: MATERI-BEL/I/MD/008048

CE⁰¹²⁰ Mimics inPrint is a CE-marked product.

© 2022 – Materialise N.V. All rights reserved.

This document contains the basic 'Instructions for Use' of Mimics Software. Detailed instructions can be found in the 'Reference Guide'. English is the language used in the Mimics software interface.

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