MANUFACTURED BY

0120

Materialise NV
Technologielaan 15
3001 Leuven
Belgium

EN

This document contains general instructions for use for Signature Planner. For detailed instructions please refer to the user manual.

DESCRIPTION

Signature Planner is a 3D environment software which allows the surgeon to accurately plan a Total or Partial Knee Arthroplasty (TKA or PKA) or a Hip Arthroplasty (THA) based on MRI or CT scans of the patient. Once the plan is approved in the planner, Materialise will manufacture patient specific knee or acetabular guides that will be used during the surgery.

INDICATIONS

The Signature Planner software is used for image processing, simulation of surgical interventions and the planning of pre-operational interventions.

The Signature Planner is used to assist in the positioning of total or partial knee or acetabular cup replacement components intra-operatively and in guiding the marking of bone before cutting.

CONTRAINDICATIONS

None

WARNINGS

- The software should be operated by trained personnel, or the output should always be reviewed and approved by a qualified clinician.
- It is the clinician's ultimate obligation to exercise his/her professional judgment in any decision to follow or not follow the treatment plan made using the medical device software.
- Signature guides are to be used by a trained clinician in the performance of surgery.
- Be aware that any patient specific guides and/or anatomical models derived from a planning created in the software have been manufactured based on CT/MRI scans of the patient. If the patient's anatomy has changed significantly since the time of the CT/MRI scan, the guides or models should not be used.

PRECAUTIONS

- It is advised to use the guide or model within 6 months of performing the CT/MRI scans on which they are based. If the patient's anatomy has changed significantly since the time of the CT/MRI-scan, the guide or model should not be used, even if the time period of 6 months has not expired.

INSTRUCTIONS FOR USE

- Install
- Close all other applications before installing Signature Planner. Install Signature Planner by logging into your Signature Online Management System account and clicking the Signature Planning Suite Installer link which is available on the main page of your account. The installer will download and begin installation automatically. Follow the instructions on screen to install Signature Planner.
- Start
- To start Signature Planner, double click the Signature icon on your desktop or go via the Start > All Programs > Materialise Software > Signature.
- The start screen is opened when the software is started. Click Plan Options > Download Plan to download a plan from the Signature Online Management System which is available for approval. If you have a plan file saved locally on your PC you can open it via Plan Options > Load Plan or by dragging and dropping it into the Signature window.
- Once your plan is opened use the Femur and Tibia boxes on the left and right sides to fine tune parameters of your plan. As you fine tune the 3D bone model will be updating according to the changes.

MINIMAL REQUIREMENTS

Software

Microsoft Windows XP SP2
Adobe Reader 7.0 or higher

Hardware

Intel Pentium 4 2.4GHz or equivalent
512MB RAM
Graphics card supporting 1024x768 and 24-bit color with 4MB RAM
Non-interlaced 15" color monitor
Three-button optical mouse with scroll wheel

RECOMMENDED REQUIREMENTS

Software

Microsoft Windows XP SP3, Microsoft Windows Vista or Microsoft Windows 7
Adobe Reader 7.0 or higher

Hardware

Intel Core2Duo 2.0 GHz or equivalent
2GB RAM
ATI Radeon or NVIDIA GeForce graphics card with 256MB RAM
Non-interlaced 19" color monitor or 17" LCD display
Resolution of 1280x1024 or higher
Three-button optical mouse with scroll wheel

GEMAAKT DOOR

Materialise
innovators you can count on

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